


Agilon[®]

Surgical Matrix

**STERILE UNLESS
OPENED OR DAMAGED**

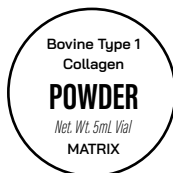

**ANTIBACTERIAL
Wound Dressing**

Product Description:

Agilon Surgical Matrix Powder is an advanced wound care device derived from 100% bovine Type 1 collagen. In the presence of wound exudate, the Agilon Surgical Matrix Powder is transformed into a soft moist wound environment. Based on in vitro performance data, Agilon Surgical Matrix exhibits broad spectrum antibacterial activity. Agilon Surgical Matrix is a biodegradable gel and settles into contact with all areas of the wound as exudate is absorbed. Agilon Surgical Matrix is sterile, single-use powder sold as a prescription-only product.

Indications For Use:

This product may be used in the management of:

- Partial and full thickness wounds
- Pressure (stage I-IV) and venous ulcers
- Ulcers caused by mixed vascular etiologies
- Venous stasis and diabetic ulcers
- 1st and 2nd degree burns
- Cuts, abrasions and surgical wounds

Directions for use:

Cleanse wound site with gentle cleanser; apply Agilon Surgical Matrix Powder in a layer approx. 1 /4" thick, do not pack tightly; cover with non-adherent dressing; repeat process as needed.

- Collagen dressing may be left in place up to 7 days.
- Frequency of application will depend on drainage.
- Collagen dressing is intended for short-term use up to 4 weeks. For long-term use, an assessment by a healthcare practitioner is recommended.

Precautions:

- Agilon Surgical Matrix should not be used on persons sensitive to bovine products.
- Discontinue use and contact a physician if excessive redness, pain, swelling, blistering, infection, no sign of healing or other unexpected symptoms occur.
- Avoid eye contact, flush thoroughly with water if this occurs.
- Discard any open or unused product.
- Store in a cool, dry storage environment, <25°C
- This product is not intended for use in procedures where product comes in contact with cerebrospinal fluid.

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.



Manufactured for:
Biogenix, LLC
18007 Sky Park Circle
Irvine, CA 92614

Catalog Number: ASM-030